


[Http://www.miracle-china.cn](http://www.miracle-china.cn)

B●nnyB●x

**Products Introduction
of the Smart-type Hairdressing
Rejuvenation Apparatus ● ●**

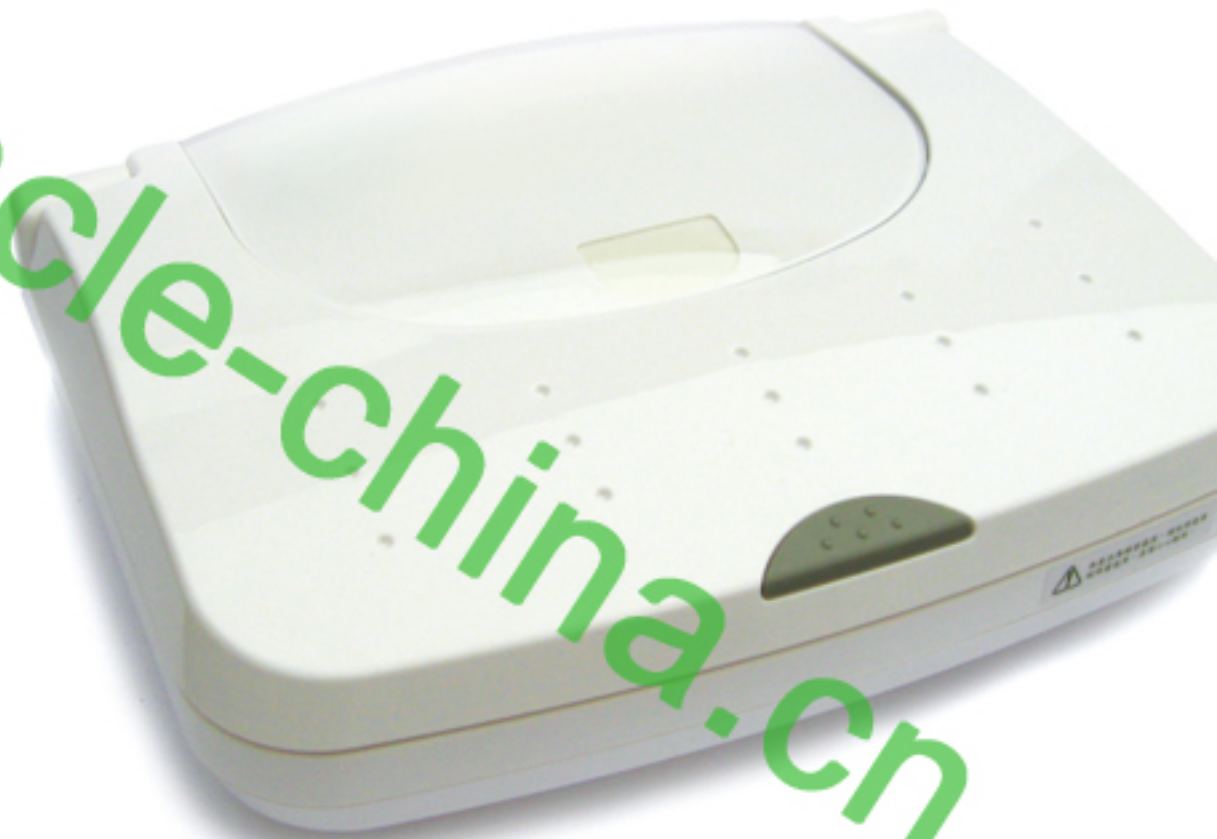

Contents

Products Introduction

- Products Overview
- Principle and Effectiveness
- Sketch Map
- Basic Equipment
- Comparative advantages

Instructions for use

- Method of Equipment Operation
- Maintenance Operation Steps
- Point Massage
- Attention

Products

Overview



<http://www.miracle-china.cn>

[Http://www.miracle-online.com](http://www.miracle-online.com)

Personified

Exquisite smart LCD screen, operation at a glance, swift and convenient, is one of the only humanity-oriented designs in the history.

High Tech

Utilizing the principle of high-power ultrasonic drive-system, treatment transducer, ultrasonic heat-massage mechanism, in accordance with the concept of ultrasound medical beauty, research and develop professional salon hairdressing apparatus which most approach to the natural function of the skin as well as the human nature.

High Efficiency

Use less than one month, revive your skin to the normal metabolism and circulation, and enhance the defense ability as well as the restoration and regeneration capability of the skin. Boast the effectiveness of moisture, tightening the skin, purifying the skin, maintaining the young dynamic nature. Your all-round professional hairdressing and body-built master.

High texture sense

Smart-type hairdressing rejuvenation apparatus, features with creative streamlines, superior and aesthetic appearance, the high technology and high-value, well-designed safe handle and high-performance transmission probe, which allows you use it more comfortable and confident.

Restore youth
is not only dream

with 10 minutes change your face,
20 minutes shape your body
and one month realize young dream



Principle & Effectiveness

Graphic representation for cell decomposition of the medical beauty:



Significant action:

- Subtle massage can improve lymphatic blood circulation;
- Increase the permeability of the cells and help nutrient transmits into as well as promotes absorption;
- Tissue cells in the deep sparks and stimulate the local metabolism.

The design of Smart-type hairdressing rejuvenation apparatus is based on the concept of ultrasound medical beauty, with 1 million high-speed weak vibration per second, rearrange the cells and narrow the gap and restore the normal absorption and metabolic functions, while generate subtle-force massage on the skin, approach to the natural function of the skin. Not only has the effect of hairdressing and body-built, but also directly transmit the health food into 4 ~ 6 Cm beneath the skin, which is the unique efficacy the general massage by hand or machine can not achieve.

Facial Beauty

Heal fine lines, refirming, desalt pigment, exfoliate the deep cutin, and reduce the generation of facial acne.

Physical beauty

Decompose the stubborn fat; improve the organization of orange peel; stimulate the release of fatty acid, make automatic consumption of the fat cells. Avoid relaxation and deformation caused by the extra floating fat in the sections of hips, internal and external sides of the legs, the small of the back, arm and external side of the calf.

Vibration Massage

Weak vibration features with 1 million times per second generate subtle-force massage on the skin, the special effects general hand or mechanical massage can not realize. Depths of the skin can produce warm feeling, the tissue 4 to 6 cm depth to the skin surface increase 0.5-1°C temperature, the action of vibration and warm heat can improve lymph blood circulation, generate normalization of the skin's metabolism, normalization, fine wrinkles gradually eliminate, dirt and waste have been ruled out successively, shape the shiny and younger skin.

High penetration

The action of vibration massage and warm-heat can achieve small massage in the cell units as well as deep of the warm-heat skin, promote blood flow and lymphatic flow, and stimulate the release of fatty acid, make the natural consumption of fat cells converting into heat energy.

Activation metabolism

Vibration function can promote the metabolism of the pigment, when the pigment continuously discharging from the epidermis and dermis, shiny skin naturally generate. Guide the skin absorb nutrition function, the use of such methods is known as (promote dermal absorption method), which can transmit the health food into the deep skin and absorb.

Body adjustments

Quickly introduce the moisture and essence ingredients into the deep skin smoothly and quickly ruled out the waste.

Sketch Map

Press button

protective cover



transformers jack

LCE LCD screen

anti-skid pads



chest maintenance key-press
(BREAST)

physical maintenance key-press
(BODY)

facial maintenance key-press
(FACE)

Facial cosmetic operation

Chest / physical
beauty operation

timing display

sound switch display

power switch

start / pause key

probe head jack

convenient slot

sound control key

facial maintenance probe

body maintenance probe

probe operation control key



Basic Equipment



- ↑ two probes
- ← One host machine
- one transformer
- ← one host machine bag
- two probe bags



E

Q

P

U

U

<http://www.miracle-china.cn>

Comparative Advantages

Smart-type hairdressing rejuvenation apparatus	General merchandise
Long time for clinical research and development, vibrating state of ultrasonic probe oscillator is stable and long life for using.	The vibration of general probe oscillator is not stable, with short use life.
Vibration frequencies of 1 million times / second, within the error of 2 percent, high stability, suitable for health food transmitting to the skin, the chest conditioning maintenance, as well as physical maintenance.	The general instability in the frequency with error up to 20 percent or more, most of the frequency range 0.3-0.6 million / second, less effective.
Special stainless steel alloy probe, perfect density and hardness of the material, there will be no corrosion cases appeared in the drug cosmetics, special handling for the mirror face of probe, smooth, attractive, equal output face, and strong conduction function.	Aluminum or plating surface probe, after use for general time (3000 times), comparatively poor conduction function as well as affect the result due to corrosion of chemical composition in the health food and unequal output face.
Manual security system of the probe, automatically stop output when nonuse, not racing equipment, can maintain constant temperature, there will be no over-temperature situation, utilize the time pause for the middle suspension other than from scratch again.	No temperature control in general, the temperature continued to rise, causing burns easily. For the machine with temperature control, the re-start time will be too long owing to the automatic cut off, with inconvenient use and high failure rate.
0.5-1W/CM2 changes output, different power output in the face, body, health probe, security, health, convenience and high satisfaction.	Generally only 0.2-0.5 W / 2, poor transmission for health food, poor performance, low-satisfaction.
Smart-type screen LCD (11.6COM * 8.4COM) displays system operation instructions, time, processes, methods panel is clear at a glance, which will facilitate your operation.	Operation requires professionals or the staffs accepted long period education.
Instructions for ending the sound.	_____
Countdown setting for treatment process in the course of operation.	_____
Creative streamlines, high-quality aesthetic appearance, combine high technology and high value.	Conventional exterior, lack of design and aesthetics
Probes can be placed in the product box, convenient and attractive.	_____

[Http://www.miracle-china.cn](http://www.miracle-china.cn)

BonnyBox

Instructions for Smart-type hairdressing
rejuvenation apparatus ● ●




Method of Equipment Operation

Beauty is a physiological problem yet is a psychological problem, which accompanies the whole life of human beings in particular for women's life, different beauty problems for different ages. Flat wart, dark sore, chloasma, skin wrinkles, senile plaque, alopecia, concave hole scars and so forth, although inflict not so much impacts on people's health and life, but the sub-health state of the skin makes many people distressed. Targeting on the problem skin, this Smart-type hairdressing rejuvenation apparatus is favored by outstanding professional beauty physicians at home and abroad.



1. Firstly insert transformer well, and press the gray section of protective cover, open the protective cover;
2. complying with the part for use, select the probe (face or body) and insert the probe wiring <probe wire-hole> connect with the host.
3. After daubing face and body with the desired use products and press <Power On / off>, and select the maintenance keys <facial part), (chest Breast), (physical Body)>, then begin Massage.
4. The diagram instruction for the operation order will be appeared in the LCD screen when use it, just follow the instructions.
5. During the operation, press <sound control key Sound On / off> can choose to play or not to play the sound.

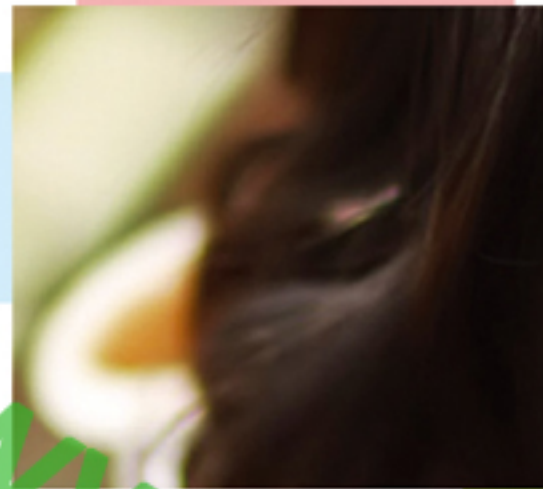

6. pressing down the <probe operational control key> during using and then immediately generate vibration. If need to stop halfway, just press <pause control key Start / Stop> will pause, if need to resume, simply click <pause control key Start / Stop> again to continue.

7. It will cut off the electricity automatically when use time comes, featuring with displaying the finish time in the liquid crystal screen, quite convenient and safe. At this point, for continued use, need to be re-boot, and then choose maintenance keys for corresponding maintenance part (facial part Face), (chest Breast), (Body), subsequently can use.

8. During operation, in case of not working, please re-check whether the transformer connect properly; confirm the connecting completion between maintenance probe wiring and the host.

<http://www.miracle-china.cn>



Maintenance

Operation Steps



<http://www.miracle-china.cn>

www.miracle-china.cn

Beauty chapter - beautiful face, youth reproduction

Combining with special gel for face and Smart-type hairdressing rejuvenation apparatus can activate the function of skin cells, repair and strengthen normal operation of the skin and show more young, healthy skin, and allows you younger than the actual age for 5-10 years, protect the traces of the years from presenting on your face.

Beauty steps




For shifting to other parts, we must press pause key firstly, then select part key for operation.

Body chapter Secluded and exquisite, wave perfect posture

Body Massage gel can be daubed according to the parts where accumulating fat easily such as the hand, buttocks, thigh and calf and apply special probe for body to help the transmission as well as accelerate the decomposition metabolism of the fat; beautify the chest lines, tighten, prevent drooping and outside expansion; also targeting on bad sitting posture, the shoulder soreness caused by the sports injuries, boasts the functions of releasing and relaxing muscle. Enable the activation for physical skills, smooth circulation, strengthen the lymph metabolism, and make the body lighter, the curve more exquisite.



Steps for Body-built

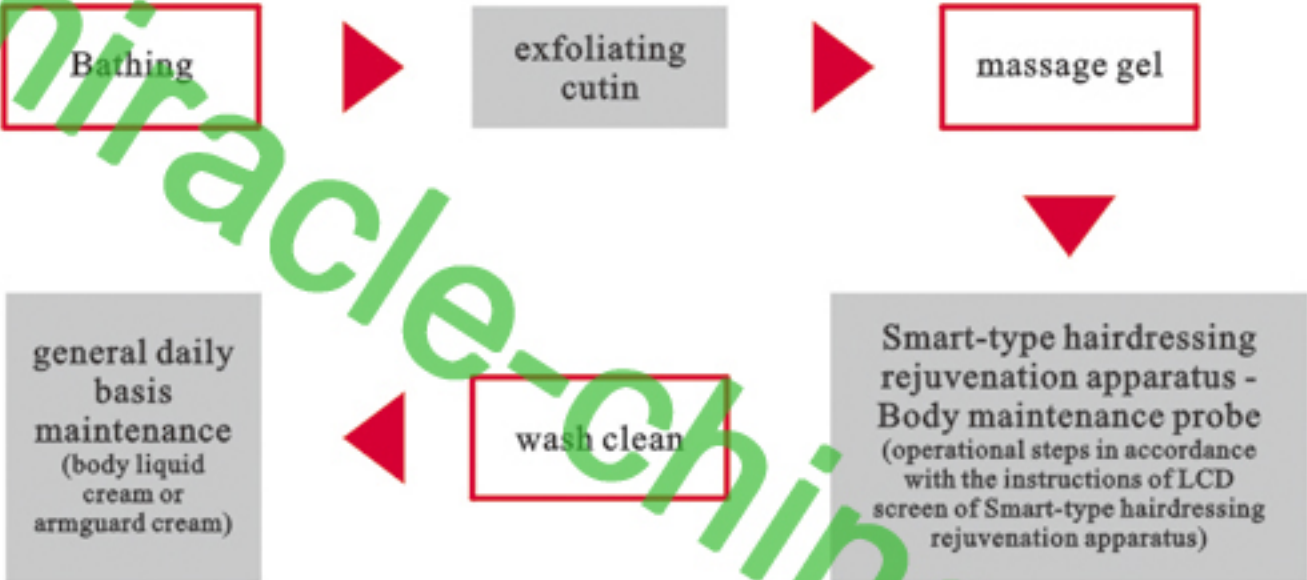

 For shifting to other parts, we must press pause key firstly, then select part key for operation.

[Http://www.miracle-china.cn](http://www.miracle-china.cn)

Point Massage

After the procedures of face and chest massage, you can press the point of the skin.

Taking advantage of Smart-type hairdressing rejuvenation apparatus to press the point directly for three seconds, unexpected effect will happen!



Attention

- The power on, the temperature of massage probe will rise slightly, this is normal phenomenon.
- Unsuitable users group:
 - Arthritis, eye diseases, patients with high-temperature heat, heart disease, skin disease, GBH patients, malignancy, high blood pressure;
 - The people with silicone, metal unidentified articles inside the skin;
 - Pregnant women, women who are under the physiological period;
 - Around the joints for the growth period juvenile;
- Scar parts after the surgery and can use after six to eight months.
- Once there is any sense of discomfort should immediately stop using and go to a doctor timely.
- This product is precision instrument, please handle with care and avoid damp.



MIRACLE CO.,LTD

Address: Room B601 Hengsheng Building B , No. 205 Huanshi Zhong Road Guangzhou China /POST NO.:510010

TEL:86-20-86673107 FAX:86-20-86672750

Email: admin@miracle-china.com

Skype: [miracle-china](#)

MSN:china.miracle@hotmail.com QQ:286372662

<http://www.miracle-china.cn>

www.miracle-china.cn