

<http://www.miracle-skin.com>



Caution:

- 1. When using weight reduction therapy, pls don't use it over 45 minutes.
- 2. During the course of weight reduction, the client will perspire a lot of sweat, pls wrap up the client in a washcloth.
- 3. When finish pls turn off the power, clean the blanket, to make that it is sanitary and durable.

Warning:

- 1. 65°C ~69°C is super high temperature.
- 2. The blanket can't be puckered after it turns on, or it will have some damage on the blanket.



Operating instructions



BEFORE USE, PLEASE READ THROUGH THIS INSTRUCTION MANUAL
THIS PRODUCT IS DESIGNED FOR HEALTHY PEOPLE ONLY.

Miracle





Remote Layer Space Fat-Losing Body-slimming

Model: SF010

1. How to use panel!

- ① Switch on 'POWER'.
- ② Press 'SELECT' button
- ③ Adjust temperature between 40-60°C.
- ④ Usage time: normally 45 minutes.
- ⑤ Press 'START' to begin or pause.

2. Introduction of all parts and power connection

- ① POWER to the general power.
- ② Press SELECT button, the indicator light of data pipes will light, indicating the admission of output.
- ③ START to the admission of output.
- ④ A, B to two temperature of areas. Press '▲ ▼' to increase and decrease the temperature.
- ⑤ The light of the data pipes at the corner right down flashes, showing that it is at work.
- ⑥ When finishing work, turn off the power of main parts, then switch off the general power.

⑦ Area A is the two corresponding outputs on the left. Area B is the two corresponding outputs on the right. When the customer only uses two outputs or on output, just connect any unit out of the five-unit body-slimming package to the two outputs of Area A and Area B. If the connection is wrong, the LED Will display □EE□, indicating outputs are disabled.

When the connection is wrong, right corner of Area A blinks, it means Area A is selected and outputs are enabled.

When the small lights in the lower right corners of Area A and Area B both blink, it means Area A and Area B are selected at the same time and outputs are enabled.

⑧ Standard power is AC 220V.50Hz or AC 240V. 60Hz.

3.FUNCTION AND PRINCIPAL

It one of the most effective and the safest scientific methods in the twenty-first century to keep fit with over-infrared-ray heat energy. All enemy in the earth will active cells under the shining by over-infrared-rays and great supercession. The over-infrared-ray owns the functions of radiation and infiltration. It can filter until 3-10cm under skins contraction, it owns the heats effects to muscles bones and inner hearts. So it has the affections of moxibustion and can speed up bloods circulation activate cells. Replenish energy and metabolism, it has good effects to reduce fatness, diminish inflammation, sterilization, force silt and treatments.

4. Caution:

- ①. When using weight reduction therapy, please don't use it over 45 minutes.
 - ②. During the course of weight reduction, the client will perspire a lot of sweat, please wrap up the client in a wash cloth.
 - ③. When finish please turn off the power, clean the blanket, to make that it is sanitary and durable.
5. Warning:
The blanket can't be puckered after it turns on, or it will have some damage on the blanket.

6. Main index of the instrument

Main index of the instrument	
Rated voltage	AC-220V 50Hz
Maximum power	600W
Main instrument size	37CM*35CM*16.5CM
Weight	2.5KG
Temperature range	25°C ~75°C



MIRACLE CO.,LTD

Address: Room B601 Hengsheng Building B , No. 205 Huanshi Zhong Road Guangzhou China /POST NO.:510010

TEL:86-20-86673107 FAX:86-20-86672750

Email: admin@miracle-china.com

Skype: miracle-china

MSN:china.miracle@hotmail.com QQ:286372662

<http://www.miracle-china.cn>

www.miracle-china.cn